

Unitron product accessory compatibility guide

Tempus platform – All technology levels

	Moxi™ RIC - All models except Moxl All	Moxi™ All RIC - All models	Stride™ BTE - All models	Insera™ ITE - All W directional models	Insera ITE - 10A W omni	Insera ITE - 312 omni	Insera ITE - 10A omni
Remote control 2	✓	--	✓	✓	✓	--	--
uStream*	✓	--	✓	✓	✓	--	--
uDirect™ 3 *	✓	--	✓	✓	✓	--	--
uMic™	✓	--	✓	✓	✓	--	--
TV Connector	--	✓	--	--	--	--	--
TV Link II/uTV™ 3 **	✓	--	✓	✓	✓	--	--
Remote Plus app	--	✓	--	--	--	--	--
uControl™ app†	✓	--	✓	✓	✓	--	--
Rechargeable kit††	✓	--	✓	--	--	--	--
IIC remote	--	--	--	--	--	--	✓†††

The Remote Plus app for Moxi All works on Apple smartphones with iOS 10.2 or newer and Android smartphones with version 6 or newer

Pairing and functionality notes

*uStream and uDirect 3 can be used for wireless streaming out of the box with no pairing needed. Must be paired to the hearing instruments to customize functionality or use as a remote.

**TV Link II and uTV 3 provide streaming and charging options for the uStream, uDirect 3 and uDirect 2

†uStream or uDirect 3 required

††Compatible with Moxi Fit and Stride M only

Programming devices

	Tempus platform	North platform	Era platform
iCube	--	--	✓ (wireless required)
iCube II	✓	✓	✓ (wireless required)
Noahlink Wireless	✓ (Moxi All only)	--	--

uControl app functionality chart

Functionality:	Introduced in version:	Accessory required:	Notes:
Remote control	v1.0	uDirect 3 or uStream	Streamer must be paired to the hearing aids in Unitron TrueFit™ fitting software and to the app/phone in order to have remote control function. Patient must be wearing streamer for remote functionality to work.
Tutorial video playback	v1.1	uDirect 3 or uStream	Streamer must be paired to hearing aids in Unitron TrueFit fitting software and to the app/phone for the app to display the correct videos. Patient does not need to be wearing the streamer to view videos.
My Ratings	v2.0	uDirect 3	Streamer must be paired to hearing aids in Unitron TrueFit fitting software, fitter must enable My Ratings in Unitron TrueFit fitting software and patient must enter activation code in uControl app to access My Ratings. Patient needs to wear uDirect 3 in order for My Ratings to capture the data logging and Log It All data with the patient comment. If the patient does not wear the uDirect 3, My Ratings will still capture the rating and comment, but no data logging or Log It All data for context.

North platform – All technology levels

	Moxi™ RIC - All models	Stride™ BTE - All models	Stride ITE - 13 W directional	Stride ITE - 312 W directional	Stride ITE - 10A W directional	Stride ITE - 10A W omni	Stride ITE - 10A omni	Stride ITE - 10A IIC
Remote control 2	✓	✓	✓	✓	✓	✓	--	--
uStream*	✓	✓	✓	✓	✓	✓	--	--
uDirect™ 3 *	✓	✓	✓	✓	✓	✓	--	--
uMic™	✓	✓	✓	✓	✓	✓	--	--
TV Link II/uTV™ 3 **	✓	✓	✓	✓	✓	✓	--	--
uControl™ app†	✓	✓	✓	✓	✓	✓	--	--
Rechargeable kit††	✓	--	--	--	--	--	--	--
IIC remote	--	--	--	--	--	--	--	✓

Pairing and functionality notes

* uStream and uDirect 3 can be used for wireless streaming out of the box with no pairing needed. Must be paired to the hearing instruments to customize functionality or use as a remote.

** TV Link II and uTV 3 provide streaming and charging options for the uStream, uDirect 3 and uDirect 2

†uStream or uDirect 3 required

††Compatible with Moxi Fit only

Era platform legacy products

Moxi² Pro/20/16/10/E, Quantum² Pro/20/16/10/E and Max 20/6/E

	Moxi ²	Moxi ² Kiss	Moxi ² Dura	Quantum ² S	Quantum ² HP	Quantum ² custom	Max™ SP	Max SPm
Smart Control	✓	--	✓	✓	✓	✓ (t-coil required)	✓	✓
Smart Alert™	--	--	--	--	--	--	✓	✓
Remote control	✓	✓	✓	✓	✓	✓ (wireless or t-coil required)	✓	✓
uDirect 2 ^{††}	✓	✓	✓	✓	✓	✓ (wireless required)	✓	✓
uDirect ^{†††}	--	--	--	--	--	--	✓	✓
uMic [†]	✓	✓	✓	✓	✓	✓ (wireless required)	✓	✓
TV Link II ^{***} /uTV 2 [†]	✓	✓	✓	✓	✓	✓ (wireless required)	✓	✓
uTV ^{***†}	✓	✓	✓	✓	✓	✓ (wireless required)	✓	✓

Pairing and functionality notes (Moxi² and Quantum²)

*uDirect 2 can be used for wireless streaming out of the box with no pairing needed. Must be paired to the hearing instruments to customize functionality or use as a remote.

**uDirect will work with Moxi² and Quantum² products in the 'generic state' out of the box; however, you cannot pair a uDirect to a Moxi² or a Quantum² instrument (no customization; no uFM program)

***TV Link II and uTV can stream to the uDirect 2 and uDirect; the charger slot only accommodates the uDirect

†Not compatible with E level products