



Operating your Unitron uStream

Your uStream can be used as a Bluetooth® device and/or as a remote control.

When you are wearing the uStream, it acts **both** as a Bluetooth device **and** as a remote control.

If you are not listening to TV or a phone call

- The large button on the front changes the aid program: 1=Everyday Listening, 2=Telephone
- The “+” and “-” buttons on the side raise and lower the volume of both aids at the same time.

If you are listening to TV

- The “+” and “-” buttons on the side raise and lower the volume of the **TV only**. You will continue to hear the sounds around you. The large button on the front **pauses** and **unpauses** the TV sound.

If you are listening to a phone call

- The “+” and “-” buttons on the side raise and lower the volume of the **caller only**. You will continue to hear the sounds around you. The large button provides advanced functions explained in the manual.

