

Operating your Unitron uDirect 3 with your Phonak TV Link S

To connect the TV Link S base and charger to your TV

- Plug the power source marked “B” into the jack marked “B” on the TV Link S base charger unit.
- Plug the small end of the red and white cable marked “A” into the jack marked “A” on the TV Link S base charger unit.
- Plug the red and white plugs into the matching red and white audio **output** jacks on the back of your TV or on the back of your **cable** or **satellite box**.

To listen to TV

- Put the uDirect 3 device around your neck.
- Turn the uDirect 3 **on** by moving the switch on the side to the on position. The green light will start to blink slowly.
- Press the **on/off** button on the TV Link S to turn the transmitter on. The green light will shortly turn solid **blue** when it connects to your uDirect 3 unit – a solid blue light will emit on your uDirect 3.
- Your hearing aids react to the uDirect 3 unit automatically. If your TV is on, your aids will automatically connect to the uDirect 3 (you’ll hear a “beep beep”) and then you will begin hearing stereo TV sound in your hearing aids.

To set the volume on your TV Link

- Turn on your TV Link S and uDirect 3 as noted above.
- Sit in your favorite TV chair and ask your spouse, family member or friend to join you and to sit where they would usually sit.
- Turn on the TV. Have the other person set the TV volume so it is just right for them.
- Then, ask the other person to stand next to the TV Link S and press the “+” and “-” buttons to turn the sound **you** hear up or down until it is just right for you. From then on the TV Link S will always transmit at the right volume for you.
- TV Link S will broadcast wirelessly to your uDirect 3 up to 100 feet away.

To turn off TV Link S and uDirect 3

- Turn off the TV Link S by pressing the on/off button and the blue light will turn off.
- Turn off the uDirect 3 by moving the switch on the side to the **off** position. When the red light turns off completely, the power is off.

To charge your uDirect 3

- Place the uDirect 3 in the charger unit. A full charge is 90 minutes and will provide 24 hours of listening time. You can use your uDirect 3 while it is charging if you wish (separate charger).

When you are using your uDirect 3, your hearing aid battery life will be shortened a bit. Remember that your aids will warn you when it is time to change the batteries and that your batteries are provided free of charge by the VA. Change your hearing aid batteries as needed.

To troubleshoot your TV Link

- Be sure your TV Link S is **off**.
- Simultaneously press and hold the “volume softer” [--] button **and** the on/off button for two seconds until the light blinks blue/green or orange/blue.
- When the light blinks blue/green or orange/blue, your TV Link S will begin transmitting pre-loaded music.
- If you hear the music, your TV Link S, uDirect 3 and hearing aids are working normally.
- If you do **not** hear the music, your audiologist can use your specific experience to determine where the problem is.

Re-pairing uDirect 3 with TV Link S

- On the TV Link S, press and hold the on button until the green light begins to rapidly blink blue. Release the button.
- Turn on the uDirect 3 and place it 3 feet from the TV Link S.
- On the uDirect 3, press and hold both the “+” button and the “<>” button at the same time until the blue light flashes rapidly. Release the buttons.
- After several seconds (may take up to 2 minutes), the blinking blue lights on **both** devices will turn solid blue. The blue light on the uDirect 3 will start to blink slowly.
- That’s it! You’re ready to go!