



Operating your uDirect neckloop


Your uDirect can be used as a Bluetooth® device and/or as a remote control.

When you are not wearing the neckloop, the device only acts as a remote control

- The large button changes the hearing aid program (1 = Everyday Listening, 2 = Telephone).
- The “+” and “-” buttons raise and lower the volume of both aids at the same time.
- The Home button on the side always returns you to your start up settings (Program 1, Everyday Listening, “optimal” volume level).

When you are wearing the neckloop, it acts as both a Bluetooth streaming device and a remote control

- If you are not listening to TV or a phone call
 - The large button changes the hearing aid program.
 - The “+” and “-” buttons raise and lower the volume of both aids at the same time.
 - The Home button on the side always returns you to your start up settings (Program 1, Everyday Listening, “optimal” volume level).
- If you listening to TV
 - The “+” and “-” buttons raise and lower the volume of the TV **only**. You will continue to hear the sounds around you. The large button **mutes** and **unmutes** the TV Sound.
- If you are listening to a phone call
 - The “+” and “-” buttons raise and lower the volume of the **call only**. You will continue to hear the sounds around you. The large button provides advanced functions explained in the manual.

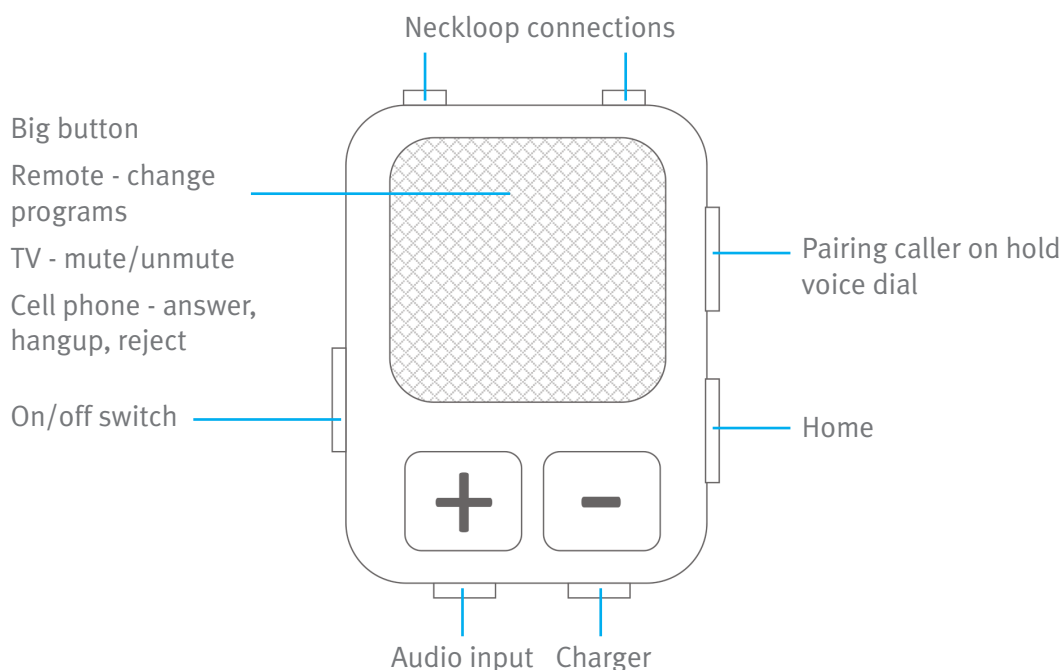

unitron™ Hearing matters