

Operating your Unitron uStream with your Phonak TV Link II

To connect the TV Link II base and charger to your TV

- Plug the power source marked “B” into the jack marked “B” on the TV Link II base charger unit.
- Plug the small end of the red and white cable marked “A” into the jack marked “A” on the TV Link II base charger unit.
- Plug the red and white plugs into the matching red and white audio **output** jacks on the back of your TV or on the back of your **cable** or **satellite box**.

To listen to TV

- Clip the uStream within 12 inches of your hearing aids. The best placement is on the collar of your shirt. Turn the uStream on by moving the switch on the side to the on position. The green light will start to blink slowly.
- Press the red on/off button on the TV Link II to turn the transmitter on. The **green** light will turn solid **blue** when it connects to your uStream unit. A blue light will turn solid blue on your uStream.
- Your hearing aids react to the uStream automatically. If your TV is on, your aids will automatically connect to the uStream (you’ll hear a “beep beep”) and then you will begin hearing stereo TV sound in your hearing aids.

To set the volume on your TV Link II:

- Turn on your TV Link II and uStream as noted above.
- Sit in your favorite TV chair and ask your spouse, family member or friend to join you and to sit where they would usually sit.
- Turn on the TV. Have the other person set the TV volume so it is just right for them.
- Then, ask the other person to stand next to the TV Link II and press the “+” and “-” buttons to turn the sound **you** hear up or down until it is just right for you. From then on the TV Link II will always transmit at the right volume for you.
- TV Link II will broadcast wirelessly to your uStream up to about 100 feet away.

To turn off the uStream and TV Link II

- Turn off the TV Link II by pressing the red on/off button and the blue light will turn off.
- Turn off the uStream by moving the switch on the side to the off position. When the red light turns off completely, the power is off.

To charge the uStream

- Place the uStream in the charger unit.
- A full charge is **two hours** which will provide four hours of listening time. You can use your uStream while it is charging if you wish (separate charger).
- When you are using your uStream, your aid battery life will be shortened a bit. Remember that your aids will warn you when it is time to change the batteries and that your batteries are provided free of charge by the VA. Change your hearing aid batteries as needed.

To troubleshoot your TV Link II

- Be sure your TV Link II is off.
- Simultaneously press and hold the “volume softer” [–] button **and** the on/off button for two seconds until the light turns blue/green or blue/orange.
- When the light turns blue/green or blue/orange, your TV Link II will begin transmitting pre-loaded music.
- If you hear the music, your TV Link II, uStream and hearing aids are working normally.
- If you do **not** hear the music, your audiologist can use your specific experience to determine where the problem lies.