

Operating your Unitron Remote Control and Remote Control 2

Your hearing aid(s) turn on automatically set at the “optimal volume” for your hearing loss. This setting is at the middle of the volume control range.

To change the program

- Press the PROGRAM CHANGE button to change the aids back and forth between any programs in your aids:
 - Everyday Listening – 1 Beep will play
 - Telephone – 2 Beeps will play

To return to start-up settings

- If you experiment with your volume or other settings and want to return to your start-up settings (“optimal volume,” Everyday Listening program), press the Home button briefly.

