


# Connecting Moxi All and Moxi All R to your cell phone

## 1 Bluetooth menu

Open the Bluetooth® menu on your phone (often found in “Settings” or “Tools”). Make sure Bluetooth is on.

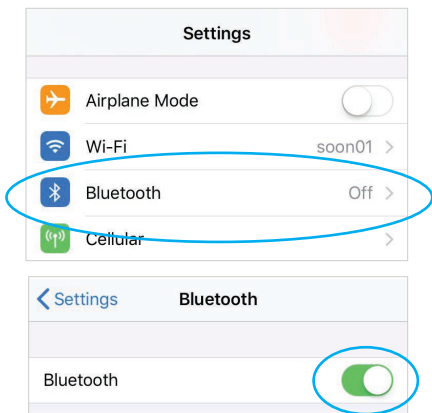


## 2 Pairing

Your hearing aid name:

---

Open and close the battery door on the (R or L) hearing aid. The name of your hearing aid will appear on the phone. Tap on it to select, and click “pair” if prompted.

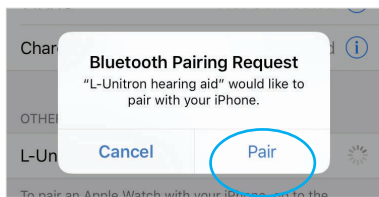


## 3 Phone calls

The microphones on the hearing aid pick up your voice so you can be hands-free. Just push either hearing aid button to accept or reject the call. **Short push to accept <2 seconds. Long push to reject >2 seconds.**





## 4 Adjust

You can increase the phone call volume by pushing the R hearing aid button, and decrease the volume by pushing the L hearing aid button. This can also be done with the volume buttons on the side of your phone.

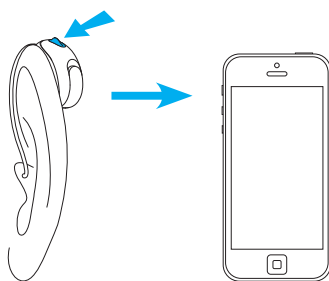


# Download the Unitron Remote Plus app

This app provides easy control of Moxi™ All settings using your smartphone. Visit the App Store (Apple™) or Google Play (Android™) and search for “Unitron Remote Plus” app.

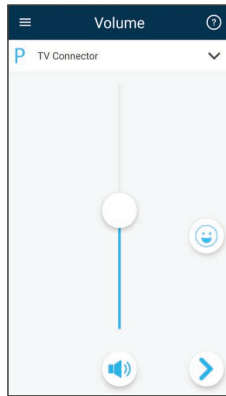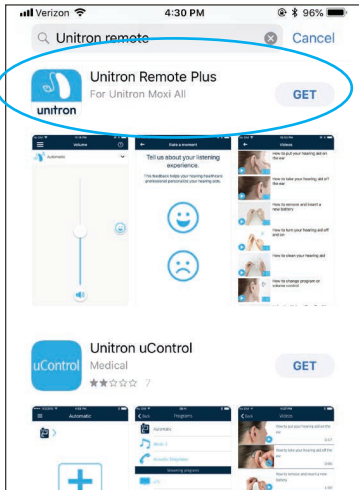

Advanced program settings

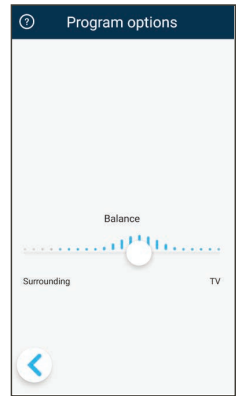

Environmental balance

Note: iOS 10.2 or Android 6.0 and Bluetooth 4.0 or newer are required to use the Unitron Remote Plus app.

## Looking for more information? Visit [unitron.com/us](http://unitron.com/us)

- Step-by-step pairing guides and videos
- Troubleshooting tips
- Frequently Asked Questions
- Information on the TV Connector and Remote Plus app accessories

Or, call us **1.866.230.9945**

The Bluetooth® word mark and logos are registered trademarks owned by the Bluetooth SIG, Inc. Apple, the Apple logo, iPhone, and iOS are trademarks of Apple Inc., registered in the U.S. and other countries. App Store is a service mark of Apple Inc. Android, Google Play, and the Google Play logo are trademarks of Google Inc.

**sonova**  
HEAR THE WORLD